



Artone communication solutions  
[www.artonecs.com](http://www.artonecs.com)

# USER MANUAL



**Artone MIC**  
Wireless Microphone

Thank you for choosing Artone.

Artone MIC is a wireless microphone that enables clear sound transmission over distance.

Artone MIC enhances understanding over distance in situations of high ambient noise, transmitting the speaker's voice directly to the user's hearing aid or Cochlear implant via Artone streamer.

Hearing aid user and Cochlear implant users can use Artone MIC in any environment to enhance the function of their hearing device and understand clearly in loud situations, in school lectures, in group meetings or over distance.

The sound from the audio source is picked up by the wireless microphone and streamed directly to the Artone streamer, and subsequently to the user's hearing device. The sound received by the hearing device will be much easier to process, as it


will not have all the added noise that sound is picking up when traveling from distance.

The compact design and internal rechargeable battery provides long play time and versatility in use.

The Artone MIC is the ideal solution to stream clear audio signal without delay or interference.

Please be sure to read through this User Manual until the end.

## 1. Product overview

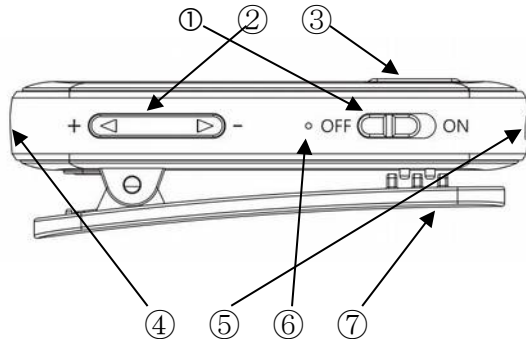

- ① Power switch
- ② Volume/sensitivity buttons
- ③ Mute/pairing button
- ④ 3.5mm Audio In Port
- ⑤ Charging Port

⑥ LED Indicator

⑦ Clips

### 1.1 What's in the box?

- 1 x Artone MIC.
- 1 x Lapel microphone (optional).
- 1 x 3.5 mm audio cable. (optional).
- 1 x USB charging cable.
- 1 x User manual.

## 1.2 Product features

Size	53 (L) x 18.3 (W) x 17 (H)mm
Weight	9.5g
Range	10 -20m in/outdoors
Working time	up to 7h
Charging time	90min
Charging voltage	5V DC
Wireless version	V 4.1
Battery capacity	90 mAh

## 1.3 Control functions

Function	Operation
Power On/Off	Move the On/OFF switch to desired position
Pairing	When switch is on, the pairing will start automatically, to force Artone MIC in to Pairing mode Press ③ Mute/pairing button.
Play / Mute	To stop listening but stay connected to the receiver, press ③ Mute/pairing button

## 1.4 LED indications

Meaning	Indicator
Power On	Blue light flashing every 1 second
Pairing mode	Red/Blue light flashing alternatively
Paired and connected	Blue light flashing every 5 seconds
Low battery – please recharge	Red light rapid flashing
Charging	Red light shining
Fully charged	Blue light shining

## 2. Using the device

### 2.1 Charging

The Artone MIC has a built-in rechargeable battery that provides long periods of operation and standby time.

1. Insert one end of Micro USB charging cable to the Artone MIC USB socket and plug the other end into another standard USB power source.

2. When using the device for the first time, charge it for minimum of 2 hours to its full capacity.

3. When the LED charging indicator turns BLUE, the Artone MIC is ready for use/ the battery is fully charged.

## 2.2 Pairing

Artone MIC is developed to work with Artone streamer. However, it can also be paired to other Bluetooth receivers.

1. Initiate the pairing sequence on your receiver (consult the instruction manual of your receiver).
2. Switch on the Artone MIC. The LEDs will shine blue and red alternatively (this process can take up to 10-20 seconds, then the pairing process is initiated).
3. When the pairing process is completed, the operation will be confirmed by a sequence of flashes of the blue LED.

## 2.3 Using the Artone MIC

### 2.3.1 Adjusting the sensitivity of the Artone MIC

Artone MIC can be used in situation of high ambient noise and quiet “classroom” environment.

- a. situations with high ambient noise (restaurant, group meeting etc.): put the Artone MIC as close as possible to the sound source (clip it to the speaker’s collar) and decrease the sensitivity on the Artone MIC to desired level. The lower the sensitivity level, the clearer the sound will be. Adjust the volume setting on your Artone streamer.
- b. quiet “classroom” environment: put the Artone MIC on a desk as close as

possible to the speaker and increase the sensitivity on the Artone MIC to desired level. The higher the sensitivity level is set, the further from the speaker the Artone MIC can be placed. Adjust the volume setting on your Artone streamer.

### **2.3.2 Using the Artone MIC with the lapel microphone (optional)**

In situations of

- outdoor activity when weather conditions (wind, rain) do not allow clear sound transmission
- when frequent adjustment of volume and sensitivity is needed
- in situations of extra high ambient noise, lowering the sensitivity of the Artone MIC to minimum and using the lapel

microphone in close distance will allow clearer voice transmission

### **2.3.3 Using the Artone MIC with the television or other audio source (optional)**

Plug in the audio cable (optional) into the 3,5mm headphone jack of your audio device (e.g. television). Clear sound from your audio source is then streamed directly into your Artone streamer and then into your hearing device.

When the audio cable is connected to the 3,5mm headphone jack of your audio device, the internal microphone of the Artone MIC will be disconnected.

## Note

- The Artone MIC will enter pairing mode automatically when powered on for the first time.
- The Artone MIC can remember previously paired devices and will attempt to pair with devices automatically, without going into pairing mode.
- Pairing mode will last 2 minutes. The receiver will enter standby mode if no devices can be connected within 2 minutes.

## 3. FAQs

**1. The Artone MIC does not turn on.**  
Please charge it.

**2. Problems with pairing to a receiver (Artone streamer).**

Turn off and then turn on both of the devices. Place the devices within 20 cm from each other and repeat the pairing process.

**3. No sound playing from the connected and paired receiver (streamer)**

Please, make sure the following:

- the Artone MIC “mute” button is not pressed
- the sensitivity setting is not at its lowest level



- the Artone streamer volume is at adequate level
- when using with hearing aid or Cochlear implant, make sure the T-Coil setting on your hearing aid or Cochlear implant is activated.

## 4. Safety Information

To avoid any damage or malfunction of the Artone MIC, follow the next rules.

1. Do not drop from high places.
2. Keep this device away from humidity, water and all other liquids.
3. Do not place or keep near any heat source.
4. Do not modify, repair or disassemble this device, especially the battery.

5. Do not place heavy objects on the device.

6. Keep the device out of reach of children.

## Warranty

**Artone**, warrants this Artone- product against defects in materials and workmanship under normal use for a period one year (or warranty period specified by the law of the country where the purchase took place) from the date of retail purchase by the original end-user purchaser ("Warranty Period"). If a product defect arises and a valid claim is received within the Warranty Period, at its consideration, Artone will either exchange the product with a new one or if possible repair the product.

The warranty does not cover abuse, deliberate damage, negligent use, and any damage caused exceeding the lines of "normal" product use. All claims must be presented with the invoice of

purchase and this document with a stamp of the retailer.

### **Warranty Limitations**

Artone limited warranty provides that, subject to the following limitations, each Product will be free from defects in material and workmanship and will conform to Artone specification for the particular Product.

### **LIMITATION OF REMEDIES YOUR EXCLUSIVE REMEDY FOR ANY DEFECTIVE PRODUCT IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT.**

Artone may elect which remedy or combination of remedies to provide in its sole discretion. Artone shall have a reasonable time after determining that a defective Product exists to repair or replace a defective Product. Artone's replacement Product under its limited warranty will be manufactured from new

and/or serviceable used parts. Artone's warranty applies to repaired or replaced Products for the balance of the applicable period of the original warranty.

### **LIMITATION OF DAMAGES**

ARTONE'S ENTIRE LIABILITY FOR ANY DEFECTIVE PRODUCT SHALL IN NO EVENT EXCEED THE PURCHASE PRICE FOR THE DEFECTIVE PRODUCT. THIS LIMITATION APPLIES EVEN IF ARTONE CANNOT OR DOES NOT REPAIR OR REPLACE ANY DEFECTIVE PRODUCT AND YOUR EXCLUSIVE REMEDY FAILS OF ITS ESSENTIAL PURPOSE.

NO CONSEQUENTIAL OR OTHER DAMAGES NOTWITHSTANDING ANYTHING ELSE IN THIS POLICY OR OTHERWISE, ARTONE WILL NOT BE LIABLE WITH RESPECT TO THE PRODUCTS UNDER ANY CONTRACT, NEGLIGENCE, STRICT LIABILITY OR OTHER LEGAL OR EQUITABLE THEORY (I) FOR ANY AMOUNT IN EXCESS OF THE PURCHASE PRICE FOR THE DEFECTIVE PRODUCT OR (II) FOR ANY GENERAL, CONSEQUENTIAL, PUNITIVE, INCIDENTAL OR SPECIAL DAMAGES. THESE INCLUDE LOSS OF LOST PROFITS, THE INSTALLATION OF REPLACEMENT PRODUCTS, AND ANY INSPECTION, TESTING, ANY DEFECT OR BY THE REPAIR OR REPLACEMENT OF PRODUCTS ARISING FROM A DEFECT IN ANY PRODUCT.

THERE ARE NO WARRANTIES WHICH EXTEND  
BEYOND THE FACE OF THE ARTONE LIMITED  
WARRANTY.

**ARTONE DISCLAIMS ALL OTHER WARRANTIES,  
EXPRESS OR IMPLIED, REGARDING THE PRODUCTS,  
INCLUDING ANY IMPLIED WARRANTIES OF  
MERCHANTABILITY, FITNESS FOR A PARTICULAR  
PURPOSE OR NONINFRINGEMENT.**

[www.artonecs.com](http://www.artonecs.com)

