

MED⁹EL



AudioLink XT

Quick Guide

AW45521.2.0 (English)

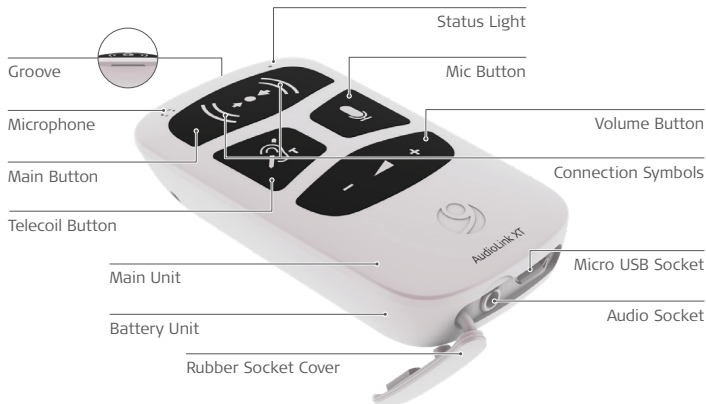


Table of Contents

AudioLink XT Overview	1	Watching TV	9
Assembling AudioLink XT	2	Using the Remote Microphone	10
Disassembling AudioLink XT	3	Using the Remote Telecoil	11
Charging AudioLink XT	4	Using an FM Receiver	12
Switching AudioLink XT On/Off	5	Indicator Lights	13
Pairing With Your Audio Processor	6	Care and Maintenance	17
Listening to Music	8	Troubleshooting	18

The AudioLink XT Quick Guide is intended as an easy-to-use reference when handling your connectivity device. For detailed information on the intended use, indications, contraindications, warnings, and precautions of AudioLink XT, or the complete information on the docking station and other accessories, please refer to the AudioLink XT user manual.

AudioLink XT



AudioLink XT

Assembling AudioLink XT



- 1 Attach the battery unit and the main unit, starting at the bottom.
- 2 Press the two parts together until the battery unit fully snaps into the main unit.
- 3 The status light will start slowly blinking to show that your AudioLink XT is ready to use.

AudioLink XT

Disassembling AudioLink XT



- 1 Insert the opening tool into the groove.



- 2 Slide the opening tool around AudioLink XT from left to right until the main unit and the battery unit detach from each other.

AudioLink XT

Charging AudioLink XT



- 1 Open the rubber socket cover.
 - 2 Plug the micro USB cable into the socket, or place AudioLink XT in the docking station. Plug the USB power adapter or docking station into a power outlet.
 - 3 The status light will be orange while charging. When the status light turns green, AudioLink XT is fully charged.
- ! AudioLink XT is delivered with the battery only partly charged. We recommend that you fully charge the battery before using AudioLink XT.

AudioLink XT

Switching AudioLink XT On/Off



- 1 To switch on AudioLink XT, press the main button for 2 seconds.
- ! AudioLink XT switches off automatically when not used for 5 minutes or not powered via USB.

- 2 The status light will illuminate in green or orange to show that AudioLink XT is switched on.
- 3 Press it again for 2 seconds to switch off AudioLink XT.

Pairing With Your Audio Processor

- 1 Switch off the audio processor.
- 2 Press the main button for approximately 2 seconds to switch on AudioLink XT and make sure it is in standby mode. Only the status light should be active.
- 3 Press the telecoil and microphone buttons at the same time for approximately 2 seconds.



- 4 AudioLink XT is now in pairing mode for approximately 60 seconds. This is indicated by continuous double-flashing of connection symbols.
- 5 Place your audio processor close to AudioLink XT and switch it on. Once the audio processor has been found, the symbol on the connected side will stop blinking.
- 6 If you are a bilateral user, repeat step 4 with your second audio processor.
- ! You can exit pairing mode at any point by pressing any button.




AudioLink XT

Listening to Music

Connect your phone or other music device to AudioLink XT, and stream music wirelessly to your audio processor.

- 1 Connect your phone or other device to AudioLink XT using the provided cables.
- 2 Select the music you would like to listen to, and press play.
- 3 The music will start streaming to your audio processor.

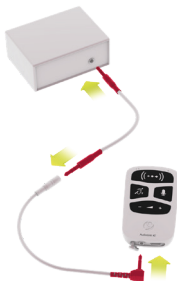

AudioLink XT

Watching TV

You can use AudioLink XT together with the docking station to hear the TV more clearly from across the room.



If your TV does not have RCA output sockets, use commercially available DAC RCA adapters.

- 1 Connect the docking station to your TV using the provided RCA cable.
- 2 Open the rubber socket cover and place AudioLink XT into the docking station.
- 3 Switch on your TV to start streaming.



Using the Remote Microphone

AudioLink XT can be used as a remote microphone, streaming sound from your environment straight to your audio processor. It can be used in both group situations and one-on-one conversations.

- 1 Press the mic button to start streaming.
- 2 The mic button will light up to show that the remote mic is activated.
- 3 To position the remote mic effectively for group conversations, you can place AudioLink XT in the docking station.
- 4 To end remote microphone mode, press the microphone button again.



AudioLink XT

Using the Remote Telecoil

AudioLink XT can be used as a remote telecoil to pick up signals from neckloops and induction loop systems to stream the sound to your audio processor.



- 1 Switch on AudioLink XT.
- 2 Press the telecoil button.
- 3 AudioLink XT is now streaming to your audio processor. This is indicated by the left/right connection symbols slowly blinking.
- 4 To exit remote telecoil mode, press the telecoil button again.

AudioLink XT

Using an FM Receiver

By connecting an FM adapter to AudioLink XT, you can connect to an FM receiver for easy access to FM systems.



- 1 Switch on AudioLink XT.
- 2 Attach the FM receiver to the FM adapter, then plug the adapter into the USB socket on AudioLink XT.
- 3 AudioLink XT is now streaming to your audio processor. This is indicated by the left/right connection symbols slowly blinking.
- 4 To stop FM receiver streaming, detach the FM Adapter from AudioLink XT.

AudioLink XT

Indicator Lights

Status Light



Light Pattern	Meaning	Required Action
 ...	AudioLink XT active	None
 ...	Charger connected and charging process finished	Remove AudioLink XT from the power outlet.
 ...	AudioLink XT active and battery level low	Charge the battery
 ...	Charger connected and battery charging	None



Left/Right Connection Symbols

Light Pattern	Meaning	Required Action
●●●●●●●●...	AudioLink XT is synchronizing to corresponding audio processor	None
▲▲▲...	AudioLink XT is streaming to corresponding audio processor	None
■■■■■■■■...	Audio processor pairing successful	None
●●—●●—●●...	AudioLink XT is searching for the audio processor	None



Mix Button

Light Pattern	Meaning	Required Action
 ...	Audio input is a mix of AudioLink XT signal and the audio processor microphones	None



Mic Button

Light Pattern	Meaning	Required Action
 ...	Remote microphone active	None

Telecoil Button



Light Pattern	Meaning	Required Action
	The remote telecoil is active	None

Mix, Telecoil, and Microphone Symbols



Light Pattern	Meaning	Required Action
	AudioLink XT reset	None
	AudioLink XT is in configuration mode	None

Care and Maintenance

AudioLink XT is designed for durability and reliability. When handled with sufficient care, it will function for a long time.

1. Do not drop the device.
2. Do not try to repair electronic parts of your AudioLink XT. Do not open, except to detach the battery, or alter any system component as this may damage the system and voids warranty claims.
3. Store AudioLink XT in a safe place when not in use.
4. Protect AudioLink XT from liquids, dust, and excessive temperatures.
5. Do not clean AudioLink XT in or under water. Use a damp cloth to gently clean AudioLink XT. Never use household cleaning products or alcohol.
6. Do not use a microwave or any other heating devices to dry AudioLink XT. If necessary, wipe it off with a dry tissue.
7. Do not use tools with sharp edges, such as a screwdriver or knife, to detach the battery from the control unit. Only use the provided opening tool.
8. Regularly charge the AudioLink XT battery to help preserve battery life.

Troubleshooting

Problem	Possible Cause	Recommended Action
Pairing does not work	AudioStream, AudioStream Adapter, or hearing aid(s) currently connected to phone	Disconnect the device(s) on your phone or switch off Bluetooth on your phone.
Audio streaming does not start. Right and/or left connection symbols on the main button do not light up	Audio processor is not paired with AudioLink XT	Pair your audio processor with AudioLink XT
No sound during audio streaming although the right and/or left connection symbols are blinking	Audio processor is out of range	– Ensure audio processor is switched on – Replace audio processor battery – Move AudioLink XT closer to audio processor

Problem	Possible Cause	Recommended Action
Distorted sound during audio streaming	Device is not within range, or there is interference from other devices	– Move AudioLink XT closer to audio processor – Remove other devices which could interfere with the wireless frequency, for example internet routers or PCs – If possible, use your Wi-Fi system's 5 GHz band
No sound, or sound too quiet	Volume too low	Increase volume on device, then on AudioLink XT
Sound too loud or distorted	Volume too high	Decrease volume on AudioLink XT, then on device

Problem	Possible Cause	Recommended Action
Streaming with the docking station or audio cables does not work although the right and/or left connections symbols are blinking	Audio cable is not connected properly	Check that all cables are properly connected, and/or that AudioLink XT is placed correctly in the docking station
AudioLink XT frozen	AudioLink XT has an unexpected problem	– Switch AudioLink XT off and on again – Press the main button and the volume – button for 7 seconds to perform a soft reset.



MED-EL Elektromedizinische Geräte GmbH
Fürstenweg 77a, 6020 Innsbruck, Austria
office@medel.com

[medel.com](https://www.medel.com)

